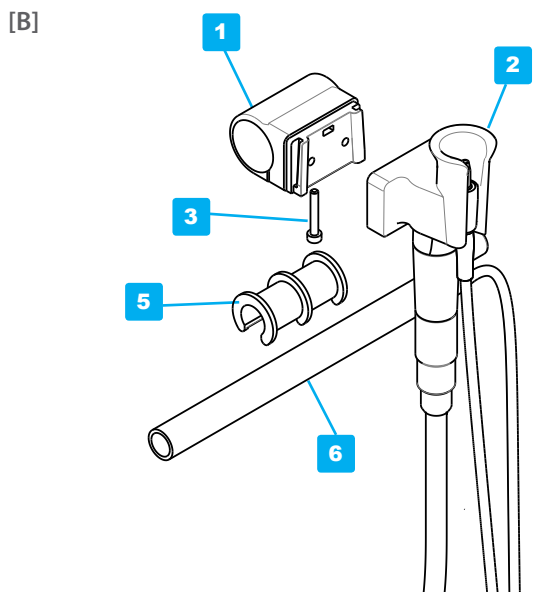
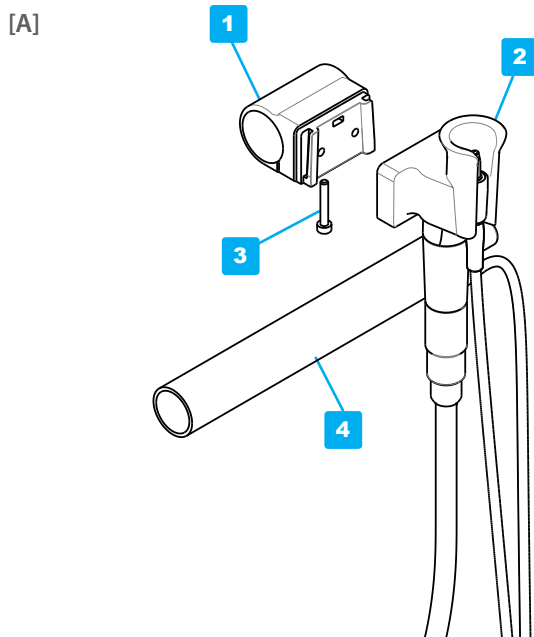
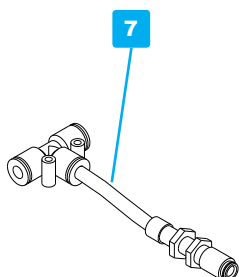
# article 0291 Tube docking holder 13-22mm

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
[C] External airpoint separately available

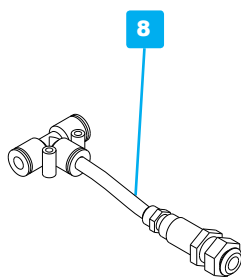Article 0377  
Quick coupling set NW female 2.7 T6

Article 0530  
Quick coupling Poly QD T6

**EN** Please read all user instructions of mirrors and handles before use.  
This adapter is to be used exclusively with the Yirro-plus® self cleaning dental mirror system

This package contains the following components:  
\* 1 Tube docking holder [1] + 1 13mm INSERT [2]  
\* 1 user instruction

**Instruction for mounting of the TUBE docking holder 13-22mm**

**[A] In case of a 22mm tube [4]:** Slide the universal docking [2] over the TUBE docking holder [1].  
Remove the bolt [3] and open the TUBE docking; place the docking holder on the 22mm tube [4]. Mount the bolt [3].

**[B] In case of a 13mm tube [6]:** Slide the universal docking [2] over the TUBE docking holder [1]. Place the white insert [5] for the 13mm tube [6].  
Remove the bolt and open the TUBE docking; place the tube docking over the insert [5] and mount the bolt [3].

**[C] Mobile use of the Yirro-plus® COMFORT system:**

The TUBE docking holder enables to 'transfer' the Yirro-plus® COMFORT system easily between 2 working rooms - Pay attention: you will need TWO external air connections: one at each delivery unit. Female air-point sets [7, 8] are available as article 0337 and 0530. A quick-coupling NW 5.0 is also available (inquire at your distributor).

For details on air-connection see separate installation sheet Yirro-plus® COMFORT system. Also available at our downloadpage: [www.yirro-plus.dhm-dental.com/downloads](http://www.yirro-plus.dhm-dental.com/downloads).

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Yirro-plus® is a registered trademark of DHM-dental B.V., the Netherlands
DHM-dental B.V.