

article 0518 MOBILE docking holder

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
[A] - [B]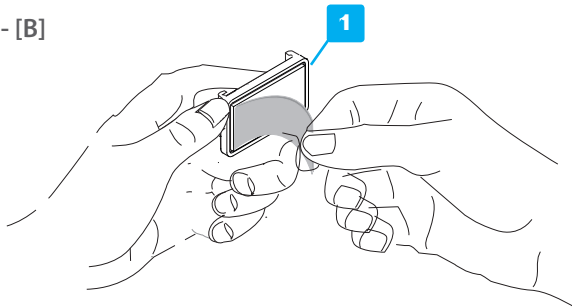

[C]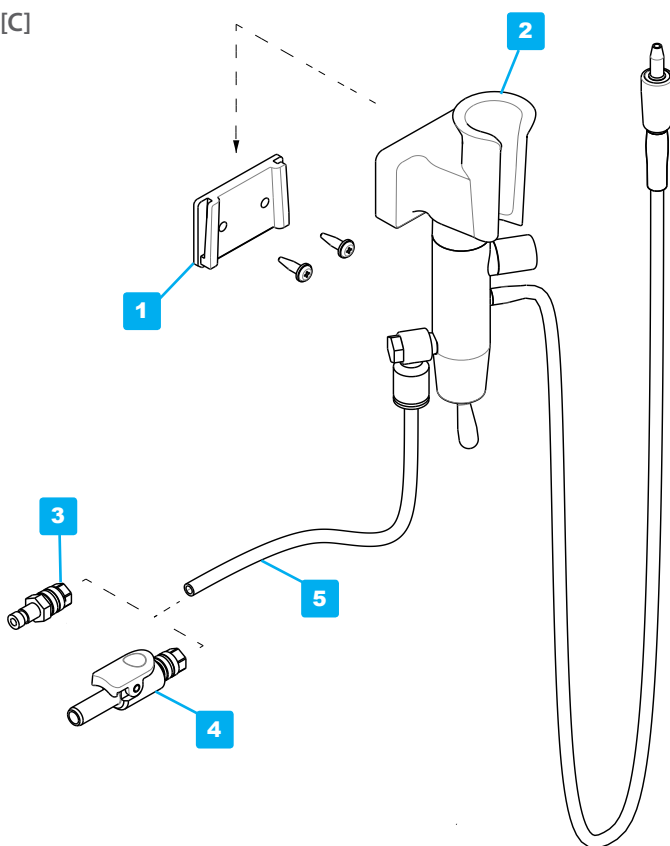

[D] External airpoint separately available

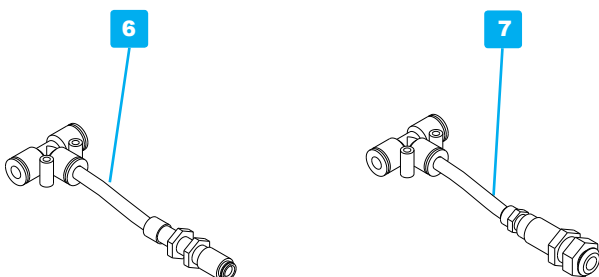Article 0377
Quick coupling set NW female 2.7 T6

Article 0530
Quick coupling Poly QD T6

EN

Please read all user instructions of mirrors and handles before use.
This mobile docking is to be used exclusively with the Yirro-plus® self cleaning dental mirror system

This package contains the following components:

- * 1 MOBILE docking holder [1] for vertical mounting
- * 2 optional screws
- * 1 user instruction

Instruction for mounting of the MOBILE docking holder on delivery unit**[A] Choose location on delivery unit**

Choose a suitable location on the delivery unit; verify if the location is in good reach and free from other instruments. Check if the 2,5m connection hose is long enough to reach the air-point from there. Clean a surface of 4x2,5 cm [1.8 x 1 inch] thoroughly by use of disinfectant.

[B] Mounting of mobile docking holder

Remove tape cover and position the MOBILE docking on desired location. Press firmly for 20 seconds. If preferred you can use 2 co-supplied screws.

[C] Mounting of universal docking in MOBILE docking holder

Click the universal docking [2] with the air regulator unit onto the MOBILE docking holder [1].

[D] Mobile use of the Yirro-plus® COMFORT system:

A MOBLIE docking holder enables to 'transfer' the Yirro-plus® COMFORT system easily between 2 working rooms - Pay attention: TWO external air connections are needed and TWO mounted MOBILE docking holders are needed; one at each delivery unit.

For details on air connection see Installation sheet COMFORT Yirro-plus® system. See also the downloadfiles: www.yirro-plus.dhm-dental.com/downloads.

External female air-point sets [7] are available as article 0337 and 0530. A male quick connector NW 5.0 is separately available (Inquire with your distributor).

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Yirro-plus® is a registered trademark of DHM-dental B.V., the Netherlands
DHM-dental B.V.