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EN

Information: Lottago® is a support for the jaw that is primarily used when a person works solo, during tartar removal or when shifts are delegated. Lottago® can be used both with the high speed suction [4] and with the low speed suction [5]. In order to use with high speed suction, the Connector [3] is needed. (Exclusively included in Lottago® 'jaw support + connector set'; not in Lottago® 'jaw support 4-pack'). Use of Connector is exclusively possible in combination with the use of 11 mm saliva ejectors [6] and then enables to disconnect the suction in between treatment, allowing the user to remove saliva etc. The Lottago® jaw support has a recommended usage time of 4 months.

User instruction: The jaw support is placed between the lower and upper parts of the jaw to allow the patient to rest the jaw during treatment. The position of Lottago® can be adjusted by moving the jaw support further in or out of the mouth. Make sure that the jaw support is placed in a position comfortable to the patient. The opening for the suction tube should not face the cheek of the patient. The position of the suction tube can be adjusted by moving it through the jaw support, but pay attention: at some point the arches will no longer protect the suction tip from the tongue and the mucous membrane. Lottago® can be used on both the right and the left side of the mouth by turning it 180 degrees. As a precaution you can mount a dental floss through the hole in order to make sure the jaw support cannot be swallowed. Dental floss should be used particularly in treatments where jaw support is not visible (i.e. with rubber dam). Carefully turn the patients' head against the Lottago to remove large amounts of saliva.

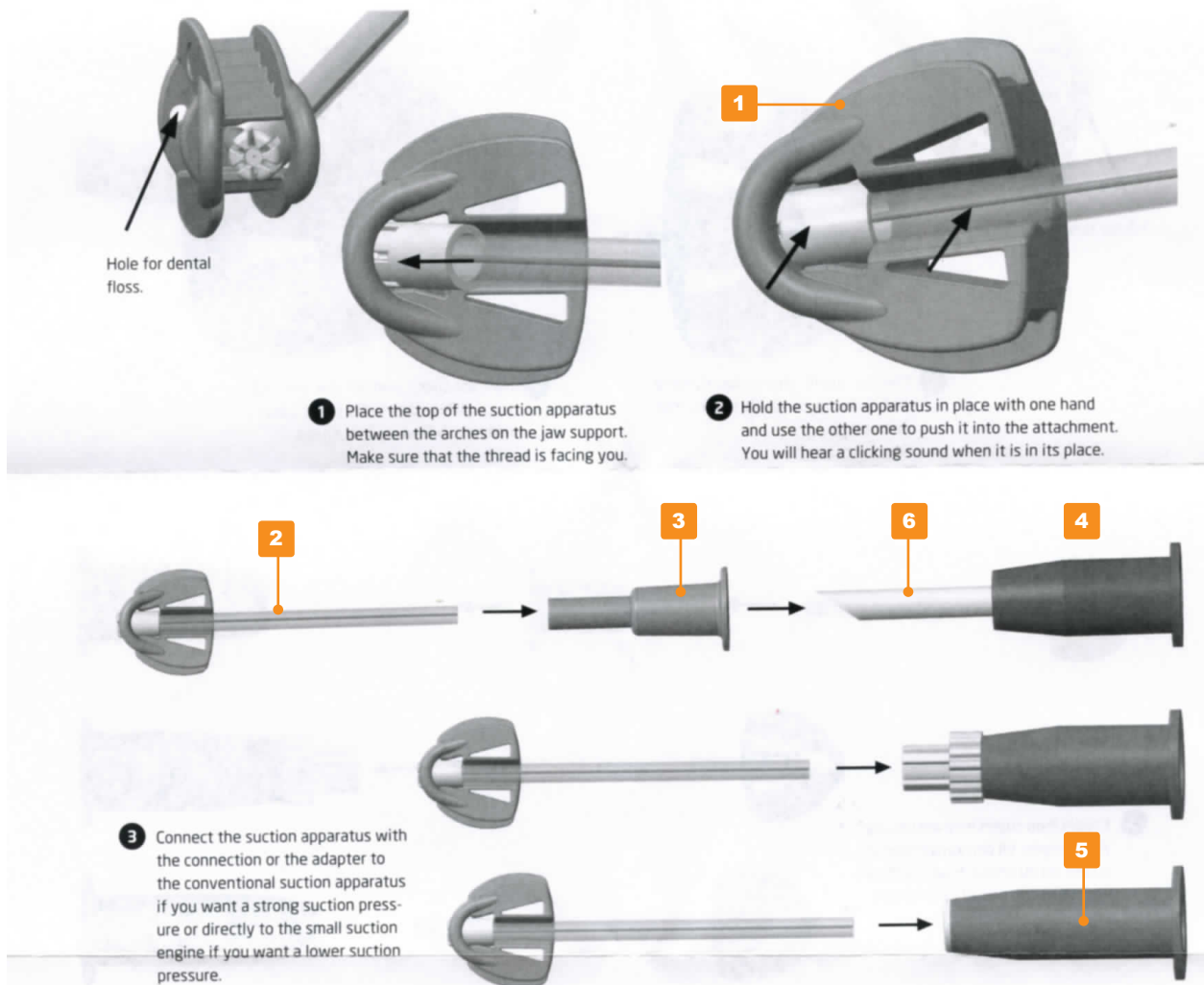
* 4 jaw supports [1] (REUSABLE)
 * 4 saliva ejectors 6mm with fixed tip [2] (SINGLE USE)
 * 1 instruction **EN**

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
WARNING!

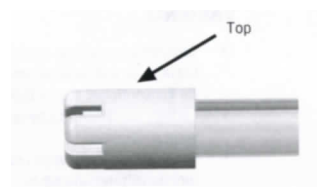- * Do not use Lottago® without a 6mm saliva ejector
- * Use Lottago® only in combination with the 6mm saliva ejector with fixed top
- * Clean Lottago® by use of thermo-disinfection prior to autoclaving between every use
- * Inspect Lottago® jaw support prior to each use and discard when visible wear appears or if the performance has changed

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