article 0002-0009 Yirro-plus® mirror 0-2

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These mirrors are to be used as a dental mirror, exclusively with Yirro-plus® handles / Read instructions before use

These mirrors are produced from durable, heat-resistant plastic

A detailed CDS protocol is available on our website: www.dhm-dental.com/downloads/CDS_protocol_2020_Yirro_V27.pdf

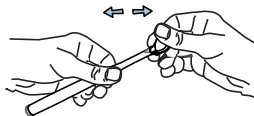
PRE-CLEANING, CHECKING & MOUNTING

Wear gloves. CLEAN mirrors and handles THOROUGHLY before use. See below.

Choose the right temperature/exposure time, suitable for plastics, for the longest possible life-time of the mirror plastic. Always check the mirror and the handle for contamination residue / damage. Worn or damaged mirrors can't be used safely and should be replaced. Mirror's containing residue do not fit properly into the handle and if a handle is damaged, the mirror head might not fit in properly.

>> Press the mirror head in the handle - AVOID TOUCHING mirror surface. Press tightly / verify sound of 'click'.

Mirrors can be polished by use of sterile soft [micro-fibre] cloth or sterilized gauze.

Disconnecting Pull the mirror head from the handle in a linear movement

CLEAN SEPARATED - NEVER CLEAN ASSEMBLED MIRRORS - AVOID ANY RISK OF CONTAMINATION AND DAMAGE.

HANDLES - MIRRORS - Cleaning by 1. thermo-disinfection or 2. ultrasonic cleaning PRIOR to additional autoclaving.

Use the Cleaning box to protect the mirror surfaces from scratches of other instruments.

AUTOCCLAVING OF MIRRORS: The plastic of the Yirro-plus mirror is resistant to autoclave temperature upto 134°C. We prescribe to use of the autoclave programme (125°C - exposure time 15 minutes) for plastic instruments. If you autoclave with 134°C, we recommend you limit exposure time to 4 minutes. Autoclaving with 134°C upto 18 minutes (PRION - Lisa) will negatively effect the lifetime of the plastic significantly. Make use of no-SPOT autoclave caps to minimize solid spotting due to unwanted particles in the autoclave system. PLEASE RINSE MIRRORS WELL PRIOR TO AUTOCLAVING. DRY mirrors with a soft micro fiber tissue prior to placing the no-SPOT cap. If you don't clean and rinse mirrors prior to autoclaving and do not use no-SPOT caps when autoclaving, this may result into solid [baked on] spots on mirror surfaces that are difficult to rub clean.

[*] STERILISATION POUCH Please consider contamination particles smaller than water damp particles are still capable to enter permeable pouch materials. Always use the no-SPOT cap, also IN a pouch.

SOFT SPOTS AFTER AUTOCLAVING? Soft spots can still occur if your autoclave is contaminated - they may also occur with use of noSPOT cap in a pouch due to residu of detergent - these spots can be easily rinsed off by use of water - afterward dry the surface by use of a [sterile] micro fiber cloth [co-sterilize it together with your mirrors] or sterilized gauze.

[*] SOLID SPOTS ON THE MIRROR SURFACE? Re-sterilize mirror while using the noSPOT cap. [if necessary re-sterilize]. After re-autoclaving, the SOLID spots will eventually loosen and transform into SOFT spots - those can be rinsed with water.

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Yirro-plus® is a registered trademark of DHM-dental B.V., the Netherlands
DHM-dental B.V.