article 0426 PREMIUM STARTSET

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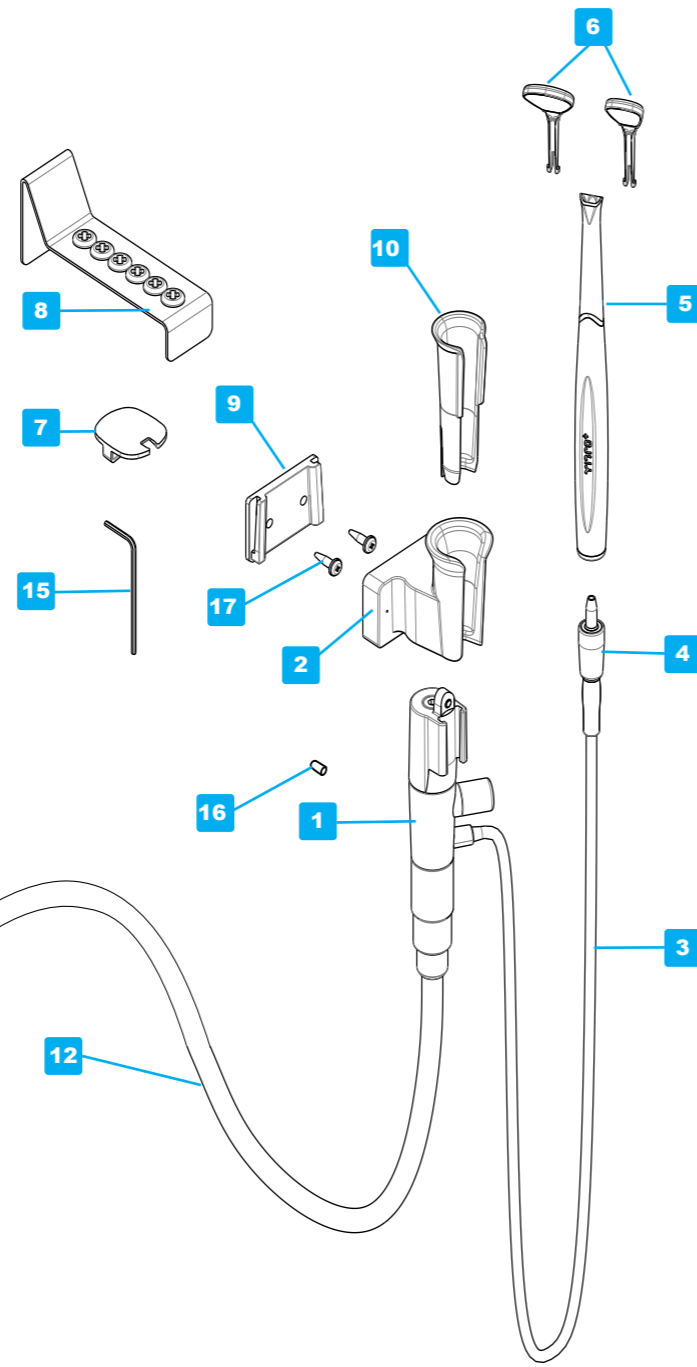
EN This package contains the following components:

- * 1 air regulator [1] with universal docking [2], 4mm silicone hose [3] and magnet coupling [4]
- * 4 handles [5]
- * 4 mirror heads [6] (2 x type no. 0 and 2 x type no. 2)
- * 4 no-SPOT autoclave caps [7] to minimize 'solid spots'
- * 1 cleaning box [8] for safe cleaning and disinfection of mirrors
- * 1 mobile docking holder [9] for vertical mounting surface
- * 1 docking adapter [10] for mounting in free slot of delivery unit
- * 1 T-part Connector 6mm and 6mm airhose [14]
- * 1 Power/Air Box [11]; connection hose [12]
- * 1 medical Power Adaptor 100-240 VAC [13] with US+EU contact blades; exchange blades for your region [other blades available on request]; contra part/wires [18]
- * 1.5mm Allen Key [15], spare 3mm set screw [16], 2 optional screws [17]
- * 1 user manual PREMIUM
- * Installation instruction
- * Tips & Tricks

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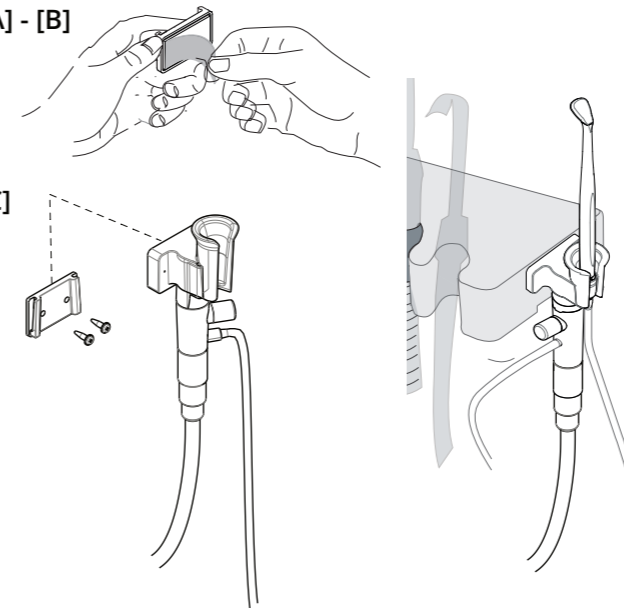
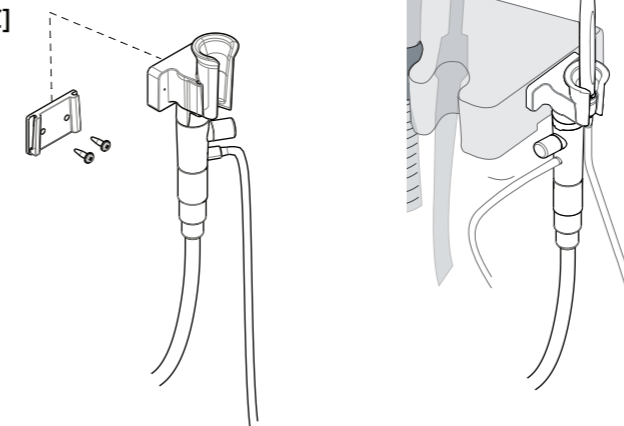
EN Mounting Option 1 - UNIVERSAL docking

[A] Choose location on delivery unit

Choose a good accessible location on the delivery unit; with enough space. Check if the length of the connection hose is able to reach an air-point from this location. Clean a surface of 4x2,5 cm [1.8 x 1 inch] thoroughly by use of disinfectant.

[B] Mounting of mobile docking holder [9]

Remove tape cover and position the mobile docking [9] on desired location. Press 20 seconds firmly. Additionally you may use the co-supplied screws [17].

[C] Mounting of docking/regulation unit in mobile docking holder

Slide universal docking [2] with air regulator unit over mobile docking holder [9]. For instructions on air and electrical connection, see other side [D/E]

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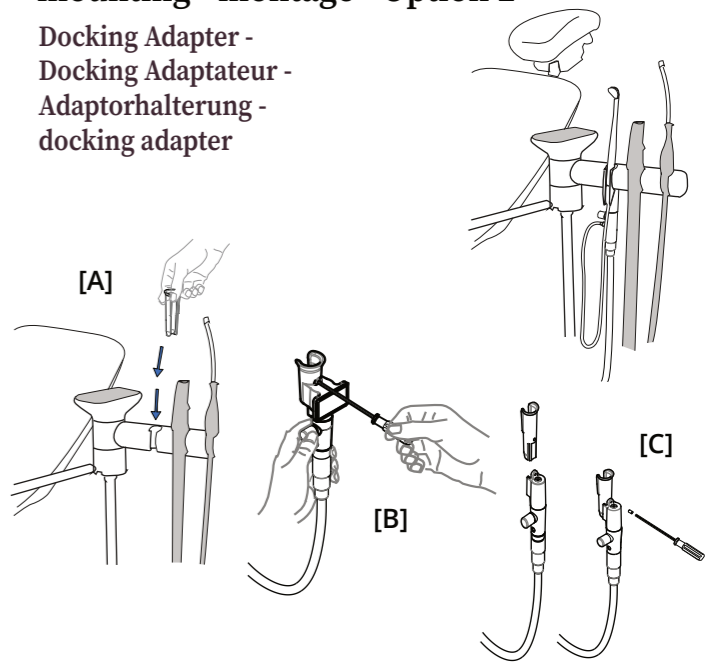
Docking Adapter - Docking Adaptateur - Adapterhalterung - docking adapter

EN Mounting Option 2 - Docking Adapter

Often, the docking adapter [10] fits in a free slot of the delivery unit. On occasion, a dedicated Integration adaptor is applicable. [See right table].

[A] Check the adapter docking

Verify the fitting of the docking adapter [10] in the spare slot and if the screw hole [16] remains accessible.

[B] Remove the Universal Docking [2]

De-screw the setscrew [16] by use of the Allen key [15]. Put set screw aside and then pull Universal Docking [2] from Regulator [1].

[C] Mount the Docking adapter

Slide the Docking Adaptor [10] onto the Regulator [1] and re-screw setscrew [16]. Place Docking adaptor with Regulator in the free slot. (For A-dec: NOTE: re-screw is only possible after placing the docking adaptor into the slot). For instructions on air connection and electrical connection, see right [D/E]

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EN Mounting Option 3 - Dedicated Integration Docking

For some situations or Units a dedicated Integration Docking is available, for instance for installation on tube, desktop or slots of certain brands (Heka, Planmeca, Sirona, XO etc. Please check the table on the right.

Dedicated Integration Dockings come with separate installation instruction.

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
Yirro-plus Integration Dockings

unit brand	included docking / adaptor	integration dockings available	article
A-DEC	YES [***]	[available 15 mm slot 99.0652.03 (A-dec), order through technician]	
AIREL	depends on delivery unit [*]	use special TUBE docking holder [#] on tube	
ANTHOS	YES		
BELMONT	YES [*] universal docking		
CART [Premium/Comfort]	YES [*] universal docking	TUBE docking holder a13/a22 mm	0291
CART [Premium]	YES [*] universal docking	MOBILE docking holder	0518
CART [Comfort]	YES [*] universal docking; extra space	MOBILE EXTENSION 16 mm docking holder	0330
CASTELLINI	YES		
CHIROMEGA	YES [*] universal docking		
DKL	YES		
DÜRR	YES	[available Dürr block 7800-070-16, order through technician]	
EPIPIX	YES	TUBE docking a13/a22 mm	0291
FINNIDENT	YES [*] universal docking		[#]
GALBIATI		C1 docking	0491
HEKA UNIC / UNICINES		HEKA UNICIS docking	0323
HEKA UNICLINE		HEKA UNICLINE [***] / UNICLINE SIDE docking	0405
KAVO	YES	[COMFORT preferably mounted at the side]	
MIDMARK		C2 docking / MIDMARK / PLANMECA slot	0517
MORITA	YES [*] universal docking		
NEW IDEM ALFA	YES	by use of Docking fixation clip	0532
OMS	depends on delivery unit	C1 docking or included adaptor docking with clip	0491 / 0532
PELTON & CRANE	depends on delivery unit	special slotadaptor [#] in development	
PLANMECA COMPACT	YES [**]	C2 docking / PLANMECA SIDE docking holder	0517 / 0464
QDENT	YES		
QUETIN	depends on delivery unit [*]	with special TUBE docking holder [#] on tube	
SIRONA SINIUS	YES [*] universal docking	the SIRONA C2/4/5+ docking	0473
SIRONA TENEO	[**]	the SIRONA C2/4/5+ docking	0473
SIRONA TENEO [****]	YES [*] universal docking	the MOBILE EXTENSION 16 mm docking holder	0330
SIRONA INTEGO		the SIRONA INTEGO docking holder	0720
SIRONA C2+/C4+/C5+	[**]	the SIRONA C2/4/5+ docking	0473
SIRONA C8+	YES [*] universal docking		
STERN WEBBER	YES [*] universal docking		
STOMADENT	YES		
TRIDAC	YES [*] universal docking	[#]	
ULTRADENT	YES [*] universal docking	[#]	
YOSHIDA	YES [*] universal docking	combine with Desktop docking holder	0582
XO FLEX	YES [*] universal docking	combine with XO-2/XO-1 docking holder	0548 / 0750

[*] to be mounted exclusively on side of delivery unit [**] included adaptors can also be used
[***] integration exclusively in spare slot [****] in case no spare slot is available [#] in development

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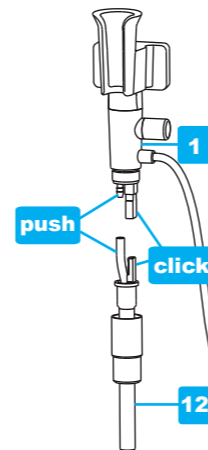EN AIR CONNECTION

[D] Connect Yirro-plus PREMIUM to compressed air

Yirro-plus® needs air pressure of 3-6 bar (45 - 90 PSI). For the connection to compressed air you can take a 6mm air line from the compressor and detach it from its connector. Mount this air line in the T-part 6mm [14] and use the small piece of 6mm air hose [14b] to mount the air line back to the compressor. Then attach the long 6mm hose [14c] to the Power/Air supply box [11]. To connect the connection hose [12] to the air-regulator [1]; mount (inner) air hose on air inlet [push] and attach the electric contacts [click]; then screw the hose connector to the regulator.

NOTE: Use of exterior air connection is not recommended for this system.

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Yirro-plus® is a registered trademark