*Beyond a mirror!*

**Yirro-plus<sup>+</sup>**

**Yirro-plus<sup>®</sup> Premium**

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This manual has been compiled with care. Nonetheless: If you have any remarks please inform us at [info@dhm-dental.com](mailto:info@dhm-dental.com)

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## EU Declaration of Conformity

EN

This declaration of conformity is issued under the sole responsibility of the manufacturer DHM-medical B.V. , located at Griftdijk 111, 6515AE Nijmegen, the Netherlands; (SRN: NL-MF-000001650). The subject of this declaration is a class 1 (MDR:2017/745) medical device with the product name "Yirro-plus", intended to be used as a **dental mirror** by professional users within the dentistry. Basic UDI-DI: 8719324509YP0426FD. The UDI-DI / REF description per system component is documented in the annexes. DHM-medical has implemented and is subject to accreditations for ISO 13485 quality management system, including GMP procedures.

DHM-medical B.V. hereby declares that:

- Yirro-plus® meets the requirements and expectations that professional users can reasonably expect from a self-cleaning mouth mirror. During the development of this device, professional users were counceled, whereby prior to the market introduction, both device and user manual have been evaluated by professional users and service technicians. The following guidelines & harmonized standards apply:

- EN 980, Symbols for use in labeling medical products
- EN 1639, Dentistry - Medical products for dentistry - Instruments
- EN 1640, Dentistry - Medical products for dentistry - Equipment
- EN-ISO 9873:2000, Dental hand instruments - mouth mirrors and handles for reuse
- EN-ISO 14971, Medical devices; risk analysis for medical devices
- EN-ISO 9168, Dentistry - Air-driven hose connectors for dental hand tools
- EN 60601-1, Medical electrical app. - Part 1: General requirements for basic safety and essential performance
- MDR: 2017 / 745

- the device to which this declaration relates is in conformity with this Regulation and, if applicable, with other relevant EU legislation providing for the issue of an EU declaration of conformity.

- the CDS protocol of the device to which this declaration relates has been validated. On this validation, no further notification is applicable.

This declaration of conformity was drawn up in Arnhem, the Netherlands, on August 30th, 2024:

P.C. Musch

DGA DHM-medical B.V.

signature:

This declaration of conformity was first issued on 1 September 2013 and is of unlimited duration as long as the conditions described in the technical documentation or the production conditions in the factory or the quality control process do not change significantly.

*Integral part of this Declaration is the document named: 'Overview Yirro-plus parts\_2024\_Aug-30-2024.pdf', in which components and -accessories of the system are visualized with the individual REF / UDI.*

**generic description**

**generic description**


Yirro-plus® system is a self-cleaning dental mirror system, a Class I medical device, intended to be used as a dental mirror during dental examination and treatment. Its clinical benefit is to provide a constant clean mirror surface during procedures.

**Product description**

This dental mirror system is developed for professional dental experts. The self-cleaning function is based on controllable air flowing over the mirror surface. The system consists of a detachable UHR mirror head (1), placed on a hollow handle (2); detachably connected to an air regulator (5) by a silicon air-hose. A detachable magnet coupling (3) is to be docked in a station (4); docking and lifting of the coupling will automatically switch ON/OFF the air flow. Below the docking, an air regulator is situated, (5) containing a control knob for air regulation. The system is to be installed independently from compressed air (3-6 bar) and electricity net by use of a connection hose (6); a Power / Air supply box (7), T-part (8) and a medical power adapter (9). Prior to use, all components are to be disinfected / sterilized by usual methods (page 11).

**the PREMIUM system**

- (1) detachable mirror head (2 sizes)
- (2) handle
- (3) magnet coupling with Ø4 mm silicone hose
- (4a) universal docking
- (4b) adapter docking
- (4c) mobile docking holder
- (5) air regulator
- (6) connection hose with strain relief
- (7) Power/Air Supply box (CE) with FDA solenoid valve 2W-NC connection to 24VDC
- (8) T-part Ø 6 mm T-part quick coupling for Ø6 mm compressed air tube
- (9) Medical Power Adapter (100-240 VAC - 50/60 Hz) out: 24 VDC according to EN 60601-1-11 with contra part and interchangeable contacts (if applicable)
- (10) Cleaning box
- (11) noSPOT autoclave caps
- (12) Allen key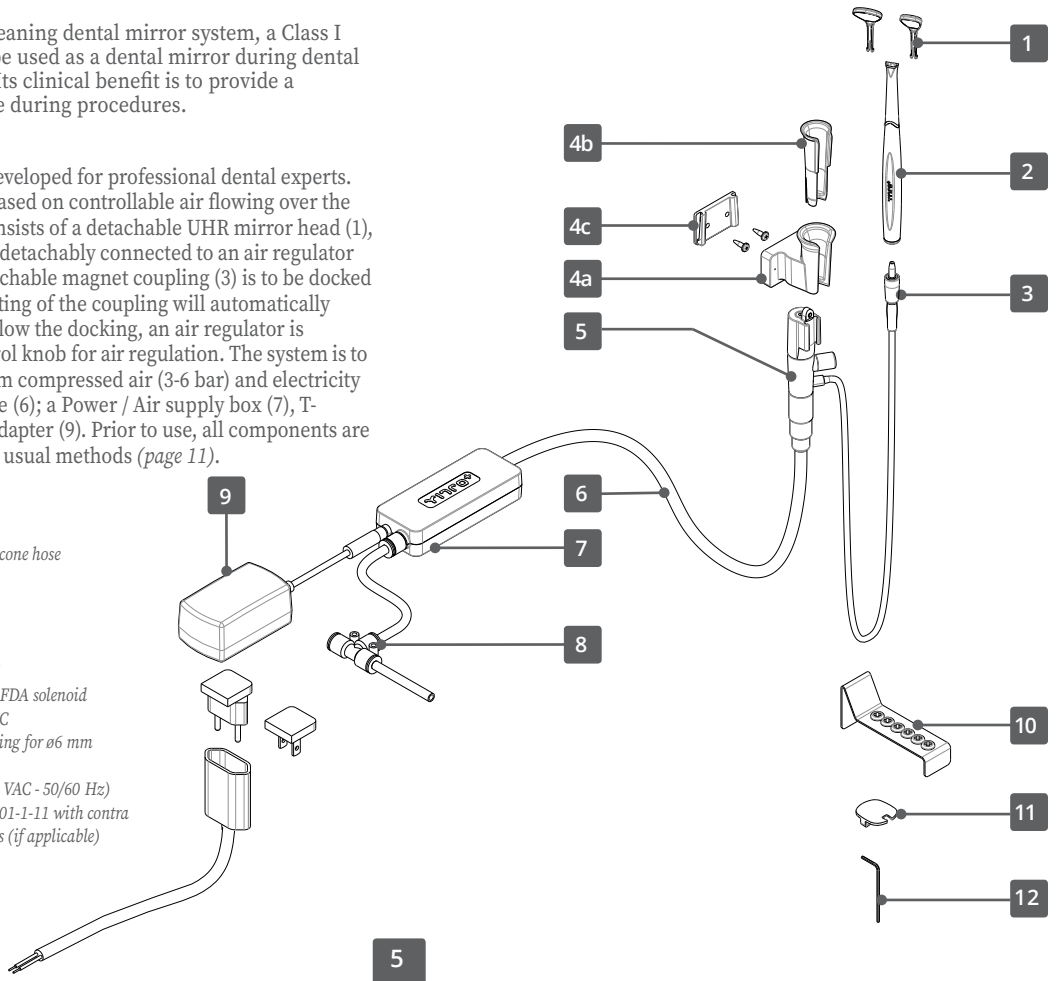

## variations

### **Yirro-plus® Premium start set**

Independent universal system, automatically switched.

### **Yirro-plus® Comfort start set**

Independent universal system, manually switched. Each start set contains: 4 handles (*durable plastic*), 4 mirror heads (2x 'no. 0'; 2x 'no. 2'), 1 cleaning box, 4 noSPOT autoclave caps, 1 magnet coupling, 1 universal docking, 1 adapter docking, 1 air regulator, connection supply and a user manual / instruction sheet.

### **mirrors and handles**

Mirror heads available in 3 sizes: no. 0 (13.2 mm); no. 2 (18.4 mm) and (separate available) no.2XL (18.4mm extra long). Mirror 0 / 2 and handle come in a 6-pack with user / cleaning instruction. Mirror 2XL comes in a 3-pack including 2XL autoclave caps. For cleaning, disinfection and sterilization, see page 11.

### **cleaning accessories**

- Cleaning box (*article 0312*) - for safe cleaning / disinfection of mirrors, offering 6 positions for mirror heads.
- no-SPOT autoclave caps (*article 0514*) - for safe sterilization of mirrors (0 / 2), to be placed on pre-cleaned mirror heads during sterilization, thus minimizing spots on the UHR mirror surface.
- nozzle adapter - (*article 0395*) for more efficient use of the thermal disinfectant pens, creating 4 positions from 1. (\*)

### **other accessories**

- Optional foot pedal (*on / off switching in between preparations - only applicable to Yirro-plus® PREMIUM system*)
- Dedicated mounting adapters are separately available (\*)

(\*) Consult our website [www.dhm-dental.com](http://www.dhm-dental.com) for most recent information on start sets; dedicated mounting adapters and all other accessories

## exclusions

Yirro-plus® is NOT intended to be used as a multi-function syringe. The air hose is not meant to be connected to water / air-suction. Yirro-plus® is not intended to treat a patient with a handle without presence of a mirror head or with a damaged mirror. The magnet coupling is NOT intended to be sterilized; it might lose its magnetism. Do not use the system on patients wearing medical devices vulnerable to magnetism.

## material specifications

### **durable, scratch-resistant and safe materials**

The Yirro-plus® device are produced from high quality heat-resistant and durable medical plastic. The mirror head contains a glass mirror with ultra-high-reflective surface. The handle contains a magnetic metal ring of surgical steel. The weight of an assembled handle with mirror is 14 grammes. Yirro-plus® air-couplings are produced from durable scratch-safe plastic; containing a magnet and a thin silicon air hose, mounted onto a metal nipple. The docking is produced from a durable plastic, resistant to manual cleaning procedures by disinfectant. The air regulator / regulation knob is assembled from preserved metal components.

### **heat-resistant, safe adhesive joints and coatings**

All adhesives and coatings that are used for the assembly of mirrors and handles are heat-resistant and absolutely safe for this application. Preferred sterilization temperature to optimize durability: 121°C - 125°C with max 15 minutes exposure.

### **reliable accessories**

Connection supply is according to relevant medical safety guidelines.

### **packaging, brochures**

Our brochures, packaging and cartons are printed on recyclable paper and cardboard.

## safety - residual risks

### safe in use

The Yirro-plus® system is a safe system provided it is handled with care and in the way it is intended to be used. Clinical evaluation tests have not shown that use of this system causes any nuisance for dentist or patient nor any gasses or chemicals that are jeopardize the health of patient or user. The dental mirror heads and handles do not contain any sharp edges that might hurt patient or user. On / off switch or regulation of the air flow has not resulted into any dangerous situations during tests.

### aerosols

During any preparation, as with all dental turbines and syringes, inevitable aerosols are spread in the air, also when using the Yirro-plus® system. For this reason you should always wear safety glasses and use a proper functioning suction system.

### noise

The sound of the use of Yirro-plus® is not louder than the sound of the regular instruments (*multi-function syringe, scaler; bur*). Please be aware that simultaneous use of instruments may generate a level of sound capable of causing hearing damage in the long term. The use of ear protection will prevent unnecessary damage.

## safety measures

### safety instructions

Only use Yirro-plus® to view your work area and clean components after every treatment by regular hygiene procedures. Avoid placing the magnet-coupling above the heart region of the patient, to eliminate any potential influence of pacemakers. Be sure to always wear ear protection, hand gloves and safety glasses and follow the cleaning protocols (*see page 11.*)

## compatibility

### compatible with any type of dental unit

Yirro-plus® PREMIUM system is a universal applicable system that can be connected to any compressor from the dental trade.


## preferred mounting

### universal mounting system


Yirro-plus® is preferably mounted at the NON-DOMINANT side, near the suction system.

### mirror hand

The system is used by the 'mirror' hand. In order to prevent the ø 4 mm silicon air hose to conflict with hoses of assistance, preferably place the Yirro-plus® docking at the non-dominant hand side, near or on the delivery or, when applicable, at the bracket-tube, near the delivery.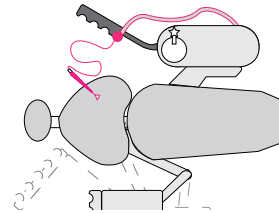

a) mounting at delivery /assistance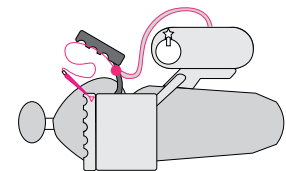

b) installation on the bracket, near the suction (tube-docking necessary) (\*)

Several options of possible locations of the docking station are shown at page 9 and on separate 'installation & mounting instructions' sheet.

(\*) Consult our website for most recent information on mounting adapters.

## quality policy & reviews

### high quality standards

Yirro-plus® is manufactured according to EN / ISO 13485 standard. Delivery of high quality is of greatest importance. In case this device should not meet your expectations, contact our customer service department; [info@dhm-dental.com](mailto:info@dhm-dental.com) or [info@dhm-medical.com](mailto:info@dhm-medical.com).

### product development

The development of this product has been concluded in cooperation with expert users. Important aspects such as ease of use, comfort, functionality, safety and hygiene were assessed in an interactive process contributing to the design, weight, user aspects, choice of materials and production of the system.

### reviews - postmarket surveys

It is our wish that Yirro-plus® is regularly reviewed by expert users. Check our social media accounts; we kindly invite you to give your feedback on the product and especially to post pictures of installation to assist us to create a database of installation examples.

## standards & regulations

The applicable standards for this product are indicated on the Declaration of Conformity, see page 4.

## mounting and installation

### choose the right spot for your docking station


Determine in advance what docking location is most applicable to your situation by studying the 3 options visualized on the right. Check the length of connection hose and the (*thin*) silicon air hose. When mounting a Yirro-plus® docking, always follow the instructions on the separate installation & mounting instruction sheet.

### installation by service technician

The Yirro-plus® PREMIUM system is to be installed independently from compressed air and electricity. Instruct your technician for the installation of an independent system by use of included components, to be connected with the main electricity and compressed-air system. More detailed installation instructions are shown on the separately included installation & mounting instruction sheet.

## location of docking station (3 options)

For the docking station the following 3 options are applicable: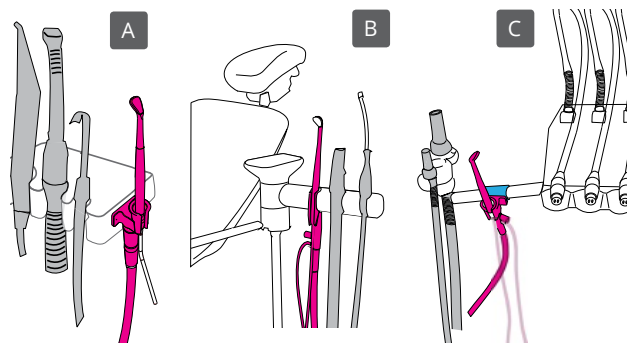

A. universal docking

B. 'in free slot'

C. on tube ø 22mm

### ad A: universal docking

On vertical flat side, the standard included universal docking can always be applied.

### ad B: 'in free slot'

If the included docking adapter does not fit into a spare slot, dedicated mounting adapters are available. (*See website for details.*)


**ad C: on tube ø13-ø22 mm**

In case of a unit with a bracket arm / suction on tubes, the Tube docking is applicable. This docking is suitable for ø22 mm tube and for ø13 mm Cart tubes.

All mounting options are further described on the separate installation & mounting instruction sheet. For most recent info see [www.dhm-dental.com](http://www.dhm-dental.com)

**placing mirror with handle**

Clean parking docking and air coupling according to standard procedures. (*Alcohol / disinfectant*). Clean/sterilize handles and mirror heads according to CDS protocol as described on p. 11.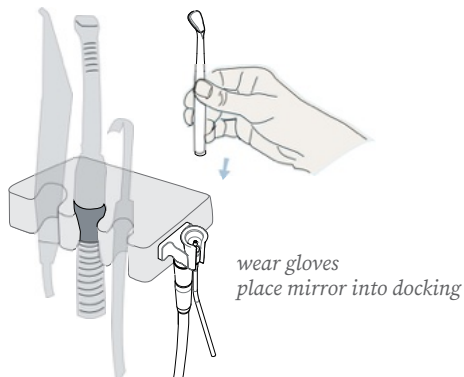


**parking (WITHOUT sleeve)**

Wear gloves and take a cleaned and sterilized dental mirror. Place it in the docking station with the mirror head upright. The magnetic air coupling will snap into the rear end of the handle, and the mirror is ready for use.

**placing dental mirrors (WITH sleeve)**

Wear gloves and take the cleaned magnetic air coupling from the parking docking. (*Air flow starts blowing*). Mount a sleeve onto the coupling by punching the air coupling through. Click the coupling at the rear end of the handle and place the assembly back into the docking station. (*Air flow will stop*).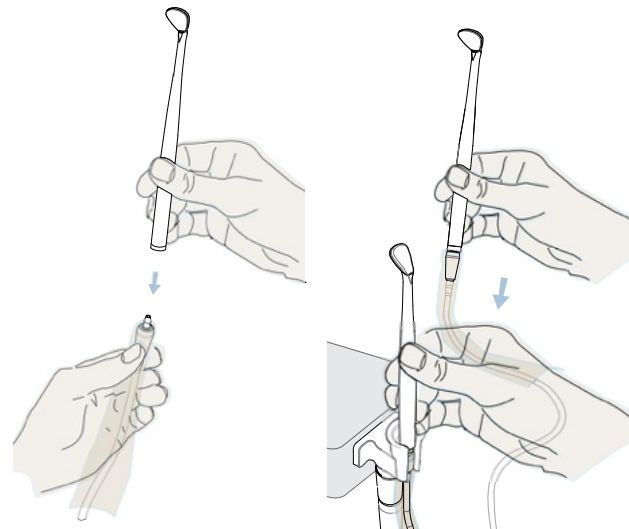

wear gloves  
mounting the sleeved coupling  
at the handle

Placing a sleeved coupling  
Into the docking station

## 'on'and 'off'switching


### docking and lifting in 1 movement

'Docking' and 'lifting' Yirro-plus® handles in and from the docking will switch 'on' or 'off' the air flow. It requires swift movements, that need a little practice, but without any further extra attention. The sensor in the system responds to the presence of the magnetic coupling.

*Conclusively, placing and lifting only the magnet coupling (without handle) will also switch the air flow.*

### lifting handle: 'ON'


Remove / lift the docked handle vertically from the docking. The air flow starts immediately.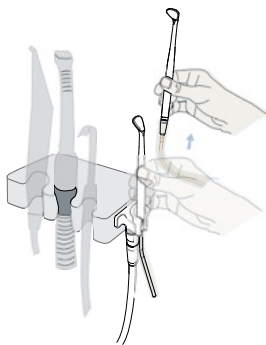

*lifting; air flow immediately starts working*

### docking: 'OFF'


Place back the dental mirror into the docking; automatically the air flow switches-OFF.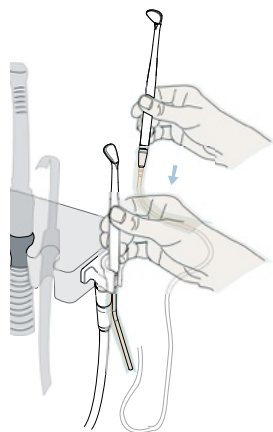

*docking; air flow immediately stops*

## air flow regulation

Set the air flow as desired, by turning the control knob. This setting can always be changed, even during treatment.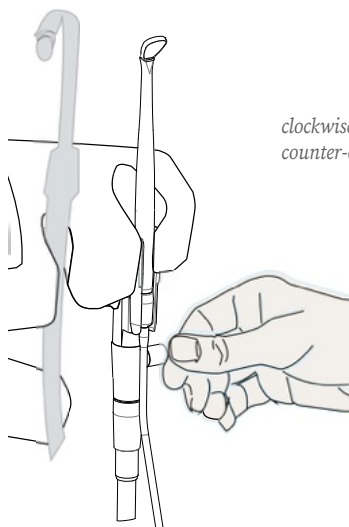

*clockwise: air flow will blow softer  
counter-clockwise: air flow will blow more firmly*

*Note:  
Keep in mind that a constant strong air flow may be uncomfortable to your patient.*

### switching ON / OFF in during a treatment

With the optional foot pedal (*article 0268*) it is possible to switch ON/OFF the air flow **during** the treatment.

Alternatively, in order to work without air flow, detach the handle from the coupling and place the coupling back in the docking (OFF); or turn the turn knob to the right (OFF).

**start using the system**

**start using the system**

For each treatment you (*and, if applicable your assistant*) follow this routine:

- 1 clean/sterilize handles and mirror heads separately (*see p.11*)
- 2 clean docking station and magnetic coupling with disinfectant, place back the coupling into the docking station
- 3 if you prefer you can use a sleeve (*see p. 10*)
- 4 check mirror head / handle for post-cleaning residu / damage and prior to assembling handle / mirror; place on dental tray or
- 5 attach cleaned dental mirror to magnet coupling and place it back into the docking station (=switch OFF)
- 6 start treatment - use dental mirror at your convenience - regulate the air by turning the control knob, if applicable
- 7 after treatment, place back dental mirror in the docking station.
- 8 after treatment: detach mirror and handle and clean separated
- 9 repeat steps 1 upto 8

**repeated use of cleaned handles and mirror heads**

The Yirro-plus® system is developed for repeated use of cleaned instruments according to hygienic rules. You should instruct your assistant in advance about the use and cleaning methods of the system. The start set provides several handles and mirror heads to conduct several treatments.

Spare handles and mirror heads are separately available:

Article 0009	6-pack no.0 mirrors
Article 0002	6-pack no.2 mirrors
Article 1062	3-pack no.2XL mirrors ( <i>incl. autocl. Cap</i> )
Article 0728	6-pack handles

**cleaning, disinfection & sterilization (CDS)**

**cleaning, disinfection & sterilization**

Yirro-plus® system consists of a mounted docking station with magnet coupling, accessories and reusable instruments. The Yirro-plus® CDS protocol consists of diverse stages. Depending on intended use (*non-critical, semi-critical or critical*) and dependent on the regulation of your region, you need to perform 1 or more stages. Check the prescribed CDS protocols in your region. (*Relevant actions are specified in action tables, see download at our website for full details. Below the main summary is provided.*)

**non-critical use**

Mounted docking station, magnet coupling and mounting accessories (*adapters*) and SOLO clips are used in a non-critical way and are leaned by manual disinfection protocol. (*Action table A(\*)*).

**semi-critical use / critical use**

Yirro-plus® mirrors / handles are reusable components and may be used in semi-critical / critical way. For cleaning, disinfection and sterilization of these components, you need to follow the applicable cleaning protocols for semi-critical / critical instruments. (*Action tables B-C-D-E (\*)*):

- Manual (pre)cleaning (*action table B*).
- Machinal cleaning by use of ultrasonic cleaner (*action table C*)
- Machine thermal disinfection (*action table D*)
- Machine sterilization by use of autoclave (*action table E*).

(\*\*) Sterilization by Eto / plasma is also possible. Sterilization by Gamma radiation / E-beam is not recommended due to negative influence on durability of plastics.

(\*\*\*) Chemical disinfection is excluded from CDS: WIP guideline (Netherlands) prescribes chemical disinfection is an exclusive option, which should exclusively be applied for instruments that are NOT resistant to thermal disinfection.

Depending if you DO or DO NOT make use of machine thermal disinfection, you need follow applicable CDS method tables:

CDS methods for cleaning WITH use of machine thermal disinfection			
component	use	CDS method	action table
docking, air-regulator, magnet coupling	non-critical use	manual disinfection	A
mirrors and handles	semi-critical use	manual (pre) cleaning	B
		(optional) ultrasonic cleaning	C
		machinal thermal disinfection	D
mirrors and handles	critical use	manual (pre)cleaning	B
		(optional) ultrasonic cleaning	C
		machinal thermal disinfection	D
		sterilization (packed or unpacked)	E

CDS methods for cleaning WITHOUT use of machine thermal disinfection			
component	use	CDS method	action table
docking, air-regulator, magnet coupling	non-critical use	manual disinfection	A
mirrors and handles	semi-critical use	manual (pre) cleaning	B
		(optional) ultrasonic cleaning	C
	AND critical use	manual cleaning/ disinfection	B
		sterilization (packed or unpacked)	E

## sterilization temperatures

Yirro-plus® handles are produced from medical plastic that is resistant to sterilization at 134°C, however, that shows a longer durability when applying plastic programm (121°C-125°C, max. 15 min. - see table E.)

### general CDS instructions

- Follow the applicable regulation for CDS of semi-critical and critical use of reusable instruments that apply to your region
- Comply to the rules for personal protection and clean workspace
- Follow the directions of this CDS protocol and ensure your staff is aware of it and can perform the instructions.
- Make use of approved cleaning agent and approved equipment
- Separate clean and dirty instruments according to the guidelines

### specific instructions for CDS of Yirro-plus®

- Disassemble mirrors/handles prior to cleaning, disinfection / sterilization.
- When rubbing dry Yirro-plus® mirror heads, it is best to use soft microfibre cloth, sterile cloth / gauze. Rub gently to avoid scratches.
- Use included cleaning accessories: Cleaning box, noSPOT Autoclave Cap.
- During cleaning / disinfection, place mirror heads in the cleaning box.
- When sterilizing mirrors in a steam autoclave, preferably use noSPOT autoclave Caps, also in the pouch. *(In order to minimize solid spots on mirror surface due to dirt in water system).*

## verify & assemble

### verify & assemble


Check with each mirror and handle for contamination residues or other damage. Worn or damaged mirrors are not safe to use and must be replaced.

### mounting

Press the mirror head in the handle - DO NOT TOUCH the mirror surface with your fingers.

*DO NOT TOUCH glass*


Press tightly and verify assembly 'click'.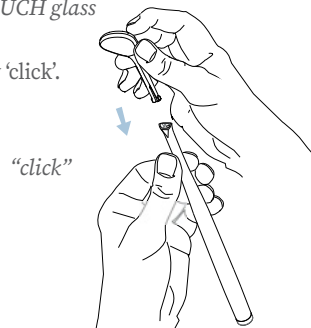

### disconnecting mirrors

#### disconnecting

Pull the mirror head from the handle by a linear movement.

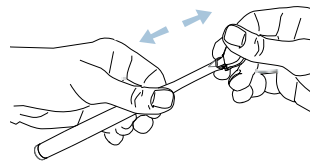*linear movement*

### storage temperature

#### storage temperature

The storage temperature of mirrors and handles is 0 °C - 50 °C.

### malfunctions

#### malfunctions

The Yirro-plus® system is produced from high-quality components and enables a carefree use for a long time. If, however, you notice a malfunction, following list will help you to make a first diagnosis:

- 1 no air flow
- 2 automatic switch does not function
- 3 the air regulation does not function
- 4 the magnet coupling does not function
- 5 the mirror head can not be mounted into the handle
- 6 silicone air hose ø 4 mm is broken or punched
- 7 the connection hose is broken or punched

#### fault 1 - no air flow

*Action:* Check if the air is completely “turned off” (control knob in most right position? Turn it leftward and check if air starts). Check if the air pressure is functioning. If this is the case and the system does not function, there might be a problem with the power supply. *Action:* Check if the contacts of the Power adapter are connected properly to power and Power / Air box. In case of a functioning compressed air system and no malfunctioning of electricity, the air might be blocked for another reason. *Contact your service technician.*

#### fault 2 - automatic switch does not function

*Action:* Make sure to place the handle deep enough in parking docking; then the sensor detects the magnet. If this does not help, check if magnetism of the magnet coupling is functioning. If the magnet is OK there may be a malfunction with the reed sensor. *Contact your service technician.*

#### fault 3 - air regulation does not function

Check if turning the control knob influences the air flow. If turning does not change the flow from hard to soft or vice versa, then the regulation system is malfunctioning. *Contact your service technician.*

#### **fault 4 - air coupling does not click into handle**

Action: Make sure the magnet is still functioning by holding it near to a metal object. If the magnet is defect, order a new coupling or contact your service technician.

#### **fault 5 - the mirror head can not be mounted into the handle**

Action: Check the front end of the handle. If it is bent, the handle cannot be used anymore. Use a replacement handle or order new ones. Check if mirror head is thoroughly cleaned; material residue (composite) may block clicking and can damage a mirror when mounted into handle. Check mirror head clickers. If they are defect, replace mirror head.

#### **fault 6 - silicone air-hose is broken or leaks**

After a period of time, the silicon hose might show some relaxation, causing the nipple-connection to leak some air. In this case pull off the air-hose and cut-off a piece (1 cm) and put it back in place. It will fit more tightly. In other cases: if the air-hose is damaged, order a spare part. You can also contact your service technician.

#### **fault 7 - the connection-hose is broken or punctured**

Action: Contact your service technician for a repair.

### **maintenance**

#### **maintenance**

Apart from regular cleaning, the Yirro-plus® system needs no maintenance. Clean mirrors and handles after every treatment by proceeding the regular hygiene procedures (page.11). Wipe /clean docking station and air coupling after every treatment with disinfectant. The regular wear and tear of mirrors and occasional wear of the silicon air hose is normal and replacement will be necessary after a certain time of use. This replacement is not to be considered as a repair and it is excluded from warranty

*REMARK 1: The docking station contains a sensor that should NOT be cleaned in thermal-disinfector or the ultrasonic cleaner, nor sterilized in the autoclave. Clean the docking by wiping (disinfectant).*

*REMARK 2: The air coupling contains a magnet, which magnetic force might vanish when sterilized in the autoclave. The magnet coupling is not used in a critical way and can be cleaned / manually disinfected.*

### **stop using**

#### **stop using**

In order to de-activate the system, switch-off the compressor and disconnect the connection hose from the air supply and dismantle the electric connection by removing the Medical Power adapter. Disconnect the T-part 6-6-6 from the air supply. Push the rig on the T- part 6-6-6 (\*) while detaching the air hose. Make sure to reconnect the ø6mm air hose to the air compressor correctly. If necessary, ask your service technician to dismantle the system and reconnect the air.

### **dismantling & waste**

#### **dismantling and waste**

The Yirro-plus® system can be dismantled into separate components for separate collection: All used and unused technical components, accessories and non-consumables are taken back for disassembly and maximal re-use of components. If you return the goods to your dealer, the dealer can periodically return the goods to the manufacturer.

*NOTE: Used consumables (like mirror heads) will NOT be taken back by the manufacturer and need to be disposed by the user.*

warranty

warranty

The Yirro-plus® system has limited 2-year warranty, which applies to installation / system components: air control system, supplied connection materials, magnetic coupling, the foot pedal and the dockings. *DHM-medical* is not liable for any consequential damage as a result of the system not functioning. Consumables such as handles, mirror heads and autoclave caps are not covered by the warranty.

environmental aspects

environmental aspects

The raw materials, production and transport processes required for production of the Yirro-plus® system, inevitably pollute the environment. By choosing sustainable materials with a long life and low energy production the environmental load of the Yirro-plus® system is minimized. The energy required for the supply of compressed air and for cleaning of the system, is a factor with impact on the environment. By using alternative energy sources such as wind or solar power, you can help minimizing the environmental aspects when using this system.

vigilance

vigilance

*DHM-medical* is convinced to have done everything possible to safely and reliably market this product. To this end, all the guidelines have been followed, discussions with experts have been held and an ISO 13485 quality management system including a GMP have been implemented. If despite all this you believe this system does not meet the qualities and standards reasonable to expect, you can report this to the appropriate authorities.

labelling of instructions

The packaging and disclaimers of the Yirro-plus® system contain the relevant product information and instruction labels / safety ratings according to EN 980:2008.

**Consult instructions for use** - The synonym for 'consult instructions for use' is 'consult operating instructions'. This symbol corresponds to the one in ISO 7000-1641 and symbol 5.3 in ISO 15223-1:2007

Symbol for **CE Mark**. This symbol certifies that this product has met European Union consumer safety, health or environmental requirements.

Symbol for **caution**. This safety symbol is used at Yirro-plus® within the context: "Attention: see Instructions for USE"

Symbol for **temperature limitation**/temperature range. Both upper and lower limits are indicated adjacent to horizontal lines.

Symbol for **non-sterile**. This symbol should be used only to distinguish between identical or similar devices sold in both sterile and non-sterile conditions.

Symbol for **Surgical steel**

Symbol for **autoclavable** at temperature indicated

Symbol for **Resistant** to chemicals of **thermal disinfection**

**Do not place** magnet coupling **on chest of people with pacemakers** or motorized insulin pumps.

Symbol for **batch code/lot number**. The symbol is accompanied by the manufacturer's batch code.

Symbol for **manufacturer**. This symbol is accompanied by address and name of manufacturer.

Symbol for **manufacturing date**. This symbol is accompanied by a date, to indicate the manufacturing date.

Symbol for **quantity per box**. Any number can be represented in the box.

Symbol for **angle of view of mirrors**. The number indicates the angle in degrees.

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